

# AeroEclipse<sup>®</sup> II BAN Patient Information

Your healthcare provider has prescribed a medication to be delivered in the **AeroEclipse<sup>®</sup> II** Breath Actuated Nebulizer (BAN). This sheet will provide you with information about the **AeroEclipse<sup>®</sup> II** BAN. If you have questions about the prescribed medication inside the nebulizer, please talk to your healthcare provider.

The **AeroEclipse<sup>®</sup> II** BAN is an innovative nebulizer that produces aerosol only when you breathe in. All other nebulizers continuously produce aerosol regardless of whether you are inhaling, exhaling or resting. As a result, when you use the **AeroEclipse<sup>®</sup> II** BAN the prescribed medication is delivered to where you need it most... deep into your lungs.

The **AeroEclipse<sup>®</sup> II** BAN features the **AeroControl<sup>®</sup>** System Regulator (SR) which provides true breath actuation for patient on-demand therapy. The **AeroControl<sup>®</sup>** SR is the green diaphragm inside the device that allows aerosol to be made when you breathe in, but not when you breathe out. Unlike other nebulizers, the **AeroControl<sup>®</sup>** SR ensures that very minimal drug aerosol escapes into the environment. You are in control of your therapy.

*The technologically advanced **AeroEclipse<sup>®</sup> II** BAN is designed to get the finest medication particles into your lungs.*

## How can the **AeroEclipse<sup>®</sup> II** BAN change your treatments?

The **AeroEclipse<sup>®</sup> II** BAN allows your healthcare provider many treatment options that may include fewer or shorter treatment times, less medication put in your nebulizer, or running the **AeroEclipse<sup>®</sup> II** BAN continuously.

*The result is better care that allows you to take your treatment as prescribed.*



# AeroEclipse<sup>®</sup> II

Breath Actuated Nebulizer

## Instructions for use

Close your lips completely over the mouthpiece and do not cover the valve on the bottom of the mouthpiece. Take slow, deep breaths; try to hold each breath for 3 to 5 seconds. The green feedback button on the top of the **AeroEclipse\*** II BAN will go down when you breathe in and will go up when you breathe out. Be sure to also follow any additional instructions your healthcare provider prescribes. Please refer to the **AeroEclipse\*** II BAN package insert for complete instructions for use.

The **AeroEclipse\*** II BAN can be used in a Continuous Mode by rotating the mode selector 90 degrees clockwise to the Continuous Mode position (▲). The nebulizer can be returned to the Breath Actuated Mode by rotating the mode selector 90 degrees counterclockwise to the Breath Actuated position (▲).

## Instructions for cleaning

Rotate the mode selector to the Breath Actuated position (▲, green feedback button is up) for both the washing and drying cycles. Between treatments, remove the supplied tubing from the bottom of the nebulizer, unscrew and remove top of nebulizer and wash nebulizer parts in a warm water and liquid dish detergent solution. Rinse in clean warm water and allow parts to dry thoroughly before reassembling.

### Please note:


The **AeroEclipse\*** II BAN is intended for single patient use. Do not share the device with another person.

- All of the components of the **AeroEclipse\*** II BAN are latex-free.
- The device should be replaced after approximately 7 days or immediately if components are cracked or the nebulizer does not function properly.

[www.aeroeclipse.com](http://www.aeroeclipse.com)

## Innovative AeroEclipse\* II BAN Features

- ① Green Feedback Button
- ② AeroControl\* System Regulator
- ③ Air Entrainment
- ④ One-Way Exhalation Valve
- ⑤ Mode Selector  
Breath Actuated (5a)  
or Continuous (5b)  
operational modes

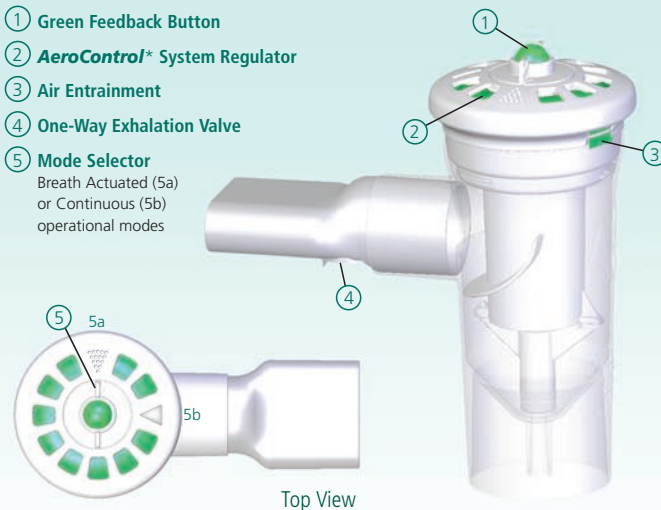

# AeroEclipse\* II

Breath Actuated Nebulizer

### Contact Information:

Trudell Medical International  
725 Third Street, London,  
Ontario, Canada, N5V 5G4  
Phone: +1-519-455-7060  
Fax: +1-519-455-7858

### Canadian Hospital Distribution:

Trudell Medical Marketing Ltd.  
758 Third Street, London,  
Ontario, Canada N5V 5J7  
Email: [tmml@tmml.com](mailto:tmml@tmml.com)  
Web site: [www.tmml.com](http://www.tmml.com)

### Authorised European Representative:

Trudell Medical International Europe Ltd.  
BioCity Nottingham, Pennyfoot Street  
Nottingham, NG1 1GF, UK  
Email: [customerservice@trudellmed.com](mailto:customerservice@trudellmed.com)

REGISTERED  
ISO 13485  
QUALITY SYSTEM



ISO CLASS 8  
CLEANROOM MANUFACTURING



Trudell Medical International<sup>1</sup>